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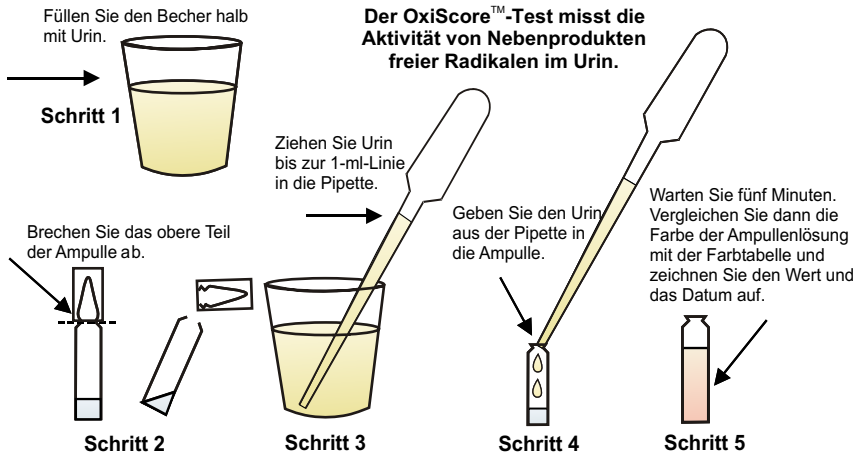
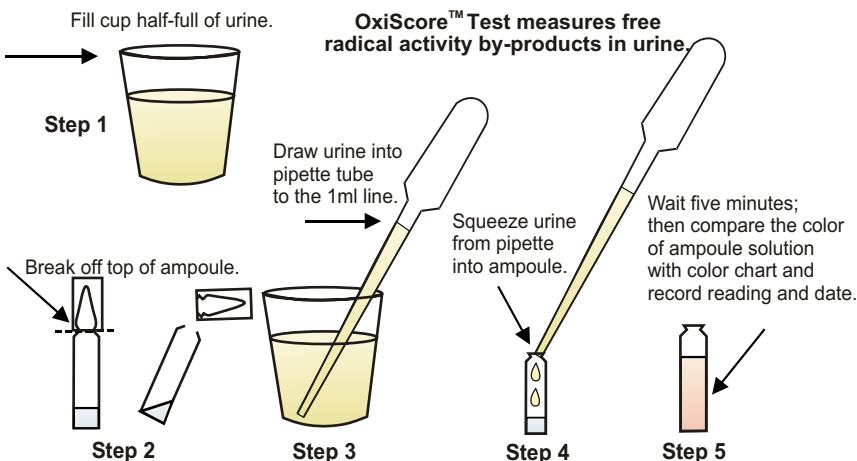
BEFORE BEGINNING: Follow pre-test protocols (included). Read this entire instruction sheet completely. Lightly tap the ampoule on the counter-top to be sure all the liquid is at the bottom of the ampoule. You may notice some crystallizing inside the ampoule. This is normal, and does not affect the outcome or accuracy of the test.

DIRECTIONS: Test First Morning Urine. Void urine in cup. Break top off of ampoule using the included plastic safety cap. Take one milliliter of urine from cup with the pipette and place into ampoule. Wait five minutes (do not let sit more than 5 minutes) and then compare the color of the ampoule solution to the color evaluation card (included in box) to find your reading. Record your reading on the card for reference.

Supplemental Dietary Restrictions Prior to Using Your Free Radical Test - Hydrate your body well the week before the test. The day before you use the test, do not take supplemental vitamins, such as vitamin C, vitamin B complex, vitamin B-1 (thiamine), vitamin B-2 (riboflavin), or vitamin B-3 (niacin), vitamin B-5 (panothenic acid), vitamin B-6 (pyridoxine), vitamin B-12 (cyanocobalamin). Any oral intake of vitamins or medication that turns your urine to an excessively yellow color may interfere with your ability to interpret the reading of your value. If you are currently taking any medications consult with your physician about the results of this test.

WARNING: Not for injection or ingestion. Do not inhale. Contents of vial are irritating and the intense dye will cause stains. Add test urine to the ampoule. **DO NOT ADD AMPOULE CONTENT TO TEST URINE. KEEP OUT OF REACH OF CHILDREN.** In case of accidental ingestion, or if the contents of the vial get in your eyes, **CALL PHYSICIAN IMMEDIATELY.** This product is not intended to diagnose, treat, or prevent any disease.